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(GB) Shiatsu foot warmer

Instruction for Use

(F) Chauffe-pieds Shiatsu

Mode d'emploi

(E) Calientapiés Shiatsu

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(I) Scaldapiedi shiatsu

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Kullanma Talimatı

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Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,
Your Beurer Team

Included in delivery: 1 Shiatsu foot warmer, 1 power supply unit, 1 operating manual.

1. About Shiatsu massage

Shiatsu is a form of body massage which was originally developed in Japan around 100 years ago. It is based on ideas anchored in traditional Chinese medicine, in particular the meridian system, i.e. the paths in the human body along which energy is conveyed.

The aim of any Shiatsu massage is to promote the physical, emotional and mental wellbeing of the person being treated. To do this, energy blocks and bottlenecks in the meridians need to be dissolved and the self-regulating ability of the body needs to be stimulated.

Shiatsu therapists achieve this by exercising pressure in flowing movements along the meridians. Although Shiatsu can be literally translated as “finger pressure”, practitioners also use the heels of their hands, their elbows and knees as well as their fingers. In the process, the therapist will use his or her full body weight in order to generate the required pressure.

Massage of a specific point on the sole of the foot may influence the corresponding organ of the body. However, we recommend studying relevant literature and asking your doctor before using point reflex zone massage to achieve the desired effect.

2. For your information

Using this electric Shiatsu foot warmer, you can provide yourself or others with a soothing, effective Shiatsu foot massage with no help required. Besides the intense Shiatsu massage, the foot warmer also provides a warming function. Both functions can be used either at the same time or separately.

3. Important safety information – read carefully and keep for later use

Failure to observe the information provided below can lead to personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information not only serves to protect your health and the health of others, it also protects the product from damage. Therefore, observe this safety information and pass on these instructions together with the product.

- Only use this footwarmer for the purpose described in the instructions for use.
- This footwarmer is not intended for use in hospitals.

- This footwarmer is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety.
- Do not use for helpless persons, small children, persons insensitive to heat (e.g. diabetics, persons with illness-related skin changes or scarred skin areas in the application area, after consuming medications or alcohol).
- Do not use the footwarmer on parts of the body which are inflamed, injured or swollen. If in doubt, seek medical advice prior to use.
- Excessively long use can cause the skin to be burned.
- Caution! Never fall asleep while the footwarmer is in operation.
- The electric and magnetic fields emitted by this footwarmer can interfere with the operation of pacemakers. However, they are far below the permissible limits: electric field strength: maximum of 5.000 V/m, magnetic field strength: maximum of 80 A/m, magnetic flux density: maximum of 0.1 millitesla. Therefore, please ask your doctor and the manufacturer of your pacemaker before using this footwarmer.
- This footwarmer
 - is only to be connected to the voltage specified on the appliance,
 - is not to be operated without supervision,
 - is not to be switched on when folded or compressed,
 - is not to be pinched between objects,
 - is not to be bent sharply,
 - is not to be used with animals,
 - is not to be used when damp.
- Only use in dry surroundings. 
- Do not pull on, twist or sharply bend the cables.
- Do not stick pins, needles or sharp objects into this footwarmer.
- This footwarmer may not be used by children unless the switch has been preset by a parent or supervisory person, or the child has been sufficiently instructed on how to use this footwarmer.
- Children should be supervised to ensure that they do not use the footwarmer as a toy.
- Use only the included power supply unit SO24 FV1200200.
- This footwarmer must be checked often for signs of wear or damage. If signs of this kind are discovered, if the cables are damaged or if this footwarmer has been used improperly, it must be taken either to the manufacturer or the dealer before further use.
- Repairs may only be carried out by specially trained personnel at a repair workshop, as special tools are required. Incorrect repairs can lead to considerable danger to the user.
- Should you still have questions on using our appliances, please contact our customer service.

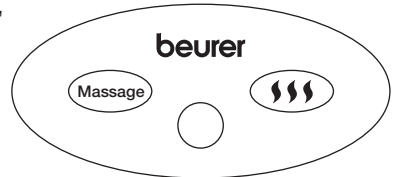
4. Proper Use

This product is intended for warming and massaging human feet. Do not use the footwarmer while wearing shoes, as this will decrease your sensitivity to heat and soil the footwarmer. This footwarmer is not intended for use in hospitals or for commercial applications. In particular, it may not be used for infants, small children, persons insensitive to heat, helpless persons or animals. The footwarmer may smell of plastic when used for the first time, but this soon stops.

5. Operation

5.1 Getting started

- Remove the packaging.
- Check the foot warmer, the power supply unit and the cable for damage.
- Connect the foot warmer to the mains by inserting the power supply cord into the middle of the control panel (to the rear of the foot warmer) and then connecting the power supply device to the mains socket.
- Ensure that the cable cannot trip anybody up.
- In order to start the Shiatsu function, press the “Massage” button on the control panel. Press the “Massage” button again in order to turn the function off.
- To start the warming function, press the “☺☺☺” button. Press the “☺☺☺” button again in order to turn the function off.
- When the warming function is turned on, a red LED lights up above the “☺☺☺” button.
- If too much pressure is exerted on the massage heads, the massage function will remain off due to safety reasons. In order to use the Shiatsu foot warmer again, briefly pull the mains supply cord from the control panel and then reinsert it.

5.2 Automatic shut-down

The foot warmer will shut-down after approx. 30 minutes for safety reasons. Press the “Massage” or the “☺☺☺” button again, if you wish to continue using the foot warmer.

Turn the foot warmer off and pull the mains supply unit from the mains socket if you do not wish to continue using the foot warmer after shut-down.

6. Cleaning and Care

The footwarmer should be kept clean, as it may not be washed.

- **Always disconnect the power plug from the socket outlet before cleaning the footwarmer.**
- Smaller stains can be removed with a cloth or damp sponge, and if necessary, some liquid detergent for delicate fabrics. The footwarmer may not be saturated in the process.
- Please note that the footwarmer may not be dried by machine, spun or dried, dry-cleaned, wrung out, pressed or ironed.
- The removable lining is machine washable at 30°C. **Do not use the footwarmer again until it is completely dry.**

7. Storage

If you do not use the footwarmer for longer periods, we recommend storing it in the original packaging in a dry place without weighing it down. Allow the footwarmer to cool down beforehand.

8. Disposal

Please dispose of the footwarmer in accordance with the Ordinance on Waste Electrical and Electronic Equipment (WEEE) 2002/96/EC. In case of inquiries, please contact the municipal authorities responsible for waste disposal in your area.

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