# beurer

## HK 56, 110, 120 XXL Gebrauchsanleitung

BEURER GmbH + Co. KG • Söflinger Str. 218 • 89077 Ulm (Germany)

Tel.: 0731 / 39 89-144 • Fax: 0731 / 39 89-255 • www.beurer.de • Mail: kd@beurer.de

- Ⓒ **Heating pad**  
Instruction for Use
- Ⓕ **Coussin chauffant**  
Mode d'emploi
- Ⓔ **Almohadas eléctricas**  
Instrucciones para el uso
- Ⓘ **Cuscino riscaldante**  
Istruzioni per l'uso

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**Dear Customer,**

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air. Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,  
Your Beurer Team

# 1. Important instructions retain for future use

- Read the information carefully.
- This appliance is not intended for use in hospitals.
- This heating pad should not be used with persons who are helpless, small children or persons insensitive to heat (e.g. diabetics, persons with skin changes related to illness or persons with scarred skin in the area where the pad is to be applied, after taking any medication to relieve pain or after taking alcohol).
- The electrical and magnetic fields emitted by this electrical product can sometimes interfere with the functioning of your heart pacemaker. However, they are far below the limits: electrical field strength max. 5000 V/m, magnetic field strength: max. 80 A/m, magnetic flux density: max. 0.1 millitesla). Please ask your doctor and the manufacturer of your pacemaker before using this product.
- Do not use to warm animals.
- The appliance must be connected only to the stipulated mains voltage.
- Do not use the heating pad in areas of the body that are inflamed, injured or swollen. If in doubt, you should seek medical advice before use.
- **Prolonged use can cause burns on the skin.**
- Caution! Never fall asleep while the heating pad is in operation.
- The heating pad must never be used folded or crumpled.
- The electronic elements in the heating pad's switch lead to slight warming of the switch during use. The switch must therefore not be on top of or covered by the heating pad when it is in use.
- Only use in conjunction with switch types specified on the heating pad.
- Do not operate the heating pad unsupervised.
- Avoid letting the device get kinked sharply.
- Do not pull on the cable, twist or bend it sharply.
- Do not pierce with needles!
- The switch and mains cable should not be exposed to moisture.
- Do not use the heating pad when it is wet.
- Check the appliance frequently for signs of wear or damage. If such signs are in evidence or if the appliance has been used incorrectly, it must be returned to the manufacturer or dealer before being used again.
- If the mains cable is damaged, it should only be replaced at a repair workshop approved by the manufacturer, as special tools are required for the repair.
- Repairs to electrical appliances must be carried out only by trained personnel. Improper repair could result in considerable risks for the user.
- When not in use, always unplug the appliance from the mains.
- Please follow the cleaning instructions (refer to instructions for cleaning and maintenance).

## 2. Use

This heating pad allows you to apply targeted heat. Heat promotes the circulation and relaxes the muscles. Our heating pads made of breathable microfleece are very adaptable and agreeable to the skin. They can also be used without their fabric cover. The detachable switch allows the pad to be cleaned in the washing machine (see cleaning instructions).

### 2.1 Switch settings

The display lights up when the heating pad is switched on.

<b>TM 56</b>	Setting: 0	Off	<b>HK 110</b>	Setting: 0	Off
	1	Minimum heat	<b>HK 120 XXL</b>	1	Minimum heat
	2	Moderate heat		2-5	Moderate heat
	3	Maximum heat		6	Maximum heat

In the beginning we recommend that you use the highest setting in order to achieve rapid heating. Later you may switch back to a lower setting.

### 2.2 Automatic switch-off function

These heating pads have an automatic switchoff function which stops the heat supply approx. 90 minutes after switching on the pad. The illuminated function display then starts to flash. To operate the heating pad again, you must first set the switch to zero. After about 5 seconds, it can be switched on again.

If the heating pad is no longer used after the timed switch-off, it should be switched off (setting "0") or else unplugged from the mains.

### 2.3 Fast heating

These heating pads have a fast heating function which allows them to heat up quickly within the first 10 minutes.

### 2.4 Additional information for HK 56

The shape of this heating pad was developed specially for use in the back and neck areas. In the neck region, the upper velcro fastener ensures that it stays in place. The rucksack fastener on the lower part of the heating pad enables you to adjust it to your body. In addition, you can change the distance of the heating pad from your body and the intensity of the heat by adjusting the strap as desired. To close the rucksack fastener, you insert the ends into one another. It can be opened again by pressing the hooks.

## 3. Cleaning and Care

These heating pads are machine-washable. Before cleaning the pad, always unplug it from the mains. Smaller stains can be removed using a cloth or moist sponge together with a little liquid detergent for delicate fabrics if necessary. Do not use any cleaning agents containing solvents on the appliance.

Please note that the heating pad should not be dry-cleaned, wrung out, machine-dried, pressed or ironed.

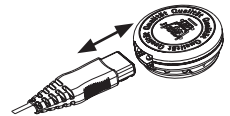
Before washing, first remove the plug-in connection together with the switch from the heating pad. Set the washing machine to a gentle washing cycle at 40 °C.

For ecological reasons, we recommend that the heating pad is only washed together with other textiles.

Use a washing powder or liquid for delicate fabrics and dose it according to the manufacturers instructions. Do not reconnect the switch to the heating pad until the plug-in connection and the heating pad are completely dry. Please note that excessive washing will wear out the heating pad. For this reason, the heating pad should only be machine-washed a maximum of 20 times throughout its entire life.

When drying the heating pad, do not use clothes pegs or anything similar, and never switch the heating pad on to dry it!

The cloth covers may be cleaned according to the symbols on the labels. Before cleaning, the cloth covers should be removed from the heating pad.

## 4. Storage

If you are not going to use the heating pad for some time, we recommend storing it in its original packaging in a dry place without weighing it down.

## 5. Disposal

Please dispose of the blanket in accordance with the directive 2002/96/EC – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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Service: BEURER GmbH + Co. KG • Söflinger Str. 218 • 89077 Ulm (Germany)