- Ⓛ **Heizkissen**  
Gebrauchsanleitung
- Ⓜ **Heating pad**  
Instruction for Use
- Ⓝ **Coussin chauffant**  
Mode d'emploi
- Ⓔ **Almohadas eléctricas**  
Instrucciones para el uso
- Ⓜ **Cuscino riscaldante**  
Istruzioni per l'uso
- Ⓣ **Isıtmalı yastık**  
Kullanma Talimatı
- Ⓡ **Электрическая грелка**  
Инструкция по применению
- Ⓟ **Poduszka elektryczna**  
Instrukcja obsługi

**Zeichenerklärung Etikett • Explanation of symbols • Légende des symboles**  
• Explicación de los símbolos • Spiegolazione dei simboli Etichetta • Etiket işaret tarifi  
• Объяснение условных знаков на этикетке • objaśnienie oznaczeń etykiet

Hinweise lesen!  
Please read the instructions!  
Lire les remarques!  
¡Leer las indicaciones!

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## ENGLISH

### Dear Customer,

We are happy that you have decided on a product from our range. Our name stands for high-quality and thoroughly tested products from the areas of warmth, blood pressure, body temperature, weight, gentle therapy, massage and air.

Please read these instructions for use carefully and follow the directions.

Best regards,  
The Beurer Team

# 1. Important safety information – read carefully and keep for later use

Failure to observe the information provided below can lead to personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information not only serves to protect your health and the health of others, it also protects the product from damage. Therefore, observe this safety information and pass on these instructions together with the product.

- Only use this heating pad for the purpose described in these instructions.
- This heating pad is not intended for use in hospitals.
- This heating pad is not intended for use by persons (including children) with restricted physical, sensory or mental capabilities and/or by persons lacking the required understanding of the equipment and how it is used, unless they are supervised by a person who is responsible for their safety.

- This heating pad should not be used with persons who are helpless, small children or persons insensitive to heat (e.g. diabetics, persons with skin changes related to illness or persons with scarred skin in the area where the pad is to be applied, after taking any pain-relieving medication or alcohol).
- Excessively long use can cause the skin to be burned.
- Caution! Never fall asleep while the heating pad is switched on.
- Do not apply the heating pad to parts of the body which are inflamed, injured or swollen. If in doubt, seek medical advice before use.
- The electric and magnetic fields emitted by this heating pad can interfere with the operation of pacemakers. However, they are far below the permissible limits: electric field strength: 5,000 V/m max., magnetic field strength: 80 A/m max., magnetic flux density: 0.1 millitesla max. Therefore, please consult your doctor and the manufacturer of your pacemaker before using this heating pad.
- Heating pad
  - only connect to the voltage specified on the appliance,
  - never operate without supervision,
  - never switch on when folded or compressed,
  - never wedge between objects,
  - never make any sharp creases,
  - never use with animals,
  - never use when damp,
  - the switch and mains cable should not be exposed to moisture.
- When in use, the electronic components in the switch of the heating pad make the switch slightly warm. The switch should therefore never be covered or placed on the heating pad when it is being operated.
- Do not pull, twist or make any sharp kinks in the cables.
- Do not stick pins or pointed objects into this heating pad.
- This appliance may not be used by children unless the switch has been preset by a parent or a supervisory person, or the child has been sufficiently instructed in how to use this heating pad safely.
- Children should be supervised to ensure that they do not use the heating pad as a toy.
- This heating pad must be checked frequently for signs of wear or damage. If such signs are present, or if the cables are damaged, or if this heating pad has been used incorrectly, it must be taken to either the manufacturer or the dealer before further use.
- Repairs may only be carried out by specially trained personnel or at an authorised repair workshop, as special tools are required. Incorrect repairs may cause considerable danger to the user.
- Should you still have questions on using our appliances, please contact our customer service.

## 2. Proper Use

This heating pad is designed for warming the human body. It allows you to apply targeted warmth. Heat promotes the circulation and relaxes the muscles. This heating pad is not intended for use in hospitals or for commercial applications. In particular, it may not be used on infants, small children, persons insensitive to heat, helpless persons or animals.

To avoid soiling or damaging the heating pad, we recommend using a fabric cover, particularly when applying essential oils and ointments.

When used for the first time, the heating pad may start to smell of plastic but this will soon go away.

## 3. Operation

### 3.1 Safety

This heating pad has a SAFETY SYSTEM. This electronic sensor technology prevents the entire surface of the heating pad from overheating by automatically switching off in the event of a fault. If the safety system switches off the heating pad automatically as a result of a fault, the temperature display on the switch is no longer illuminated even when the heating pad is switched on. Please

note that, after a fault has occurred, the back and neck heating pad can no longer be used for safety reasons and must be sent to the service address indicated.

### 3.2 Selecting temperature

The fastest way to heat up the heating pad is to first set the switch to the highest setting. A lower setting can be selected later if required.

We recommend the lowest temperature setting for extended use of the heating pad.

### 3.3 Temperature settings

- Level:
- 0 Off
  - 1 Minimum heat
  - 2 Medium heat
  - 3 Maximum heat

### 3.4 Automatic switch-off function

In heating pads with automatic cutoff, the heat supply stops approximately 90 minutes after the heating pad is operated. The illuminated display does not go out. To operate the heating pad again, you must first set the switch to the "0" position. After about 5 seconds, it can be switched on again. If the heating pad is not used after it has cut off automatically, it should be switched off ("0" position) or unplugged from the socket.

## 4. Cleaning and Care

- **Before cleaning the heating pad, always unplug it from the mains.**
- The textile cover can be cleaned according to the symbols on the label, and for this purpose must first be removed.
- Please note that the actual heating pad may not be machine-washed, spun or machine dried, chemically cleaned, wrung out, pressed or ironed.
- Smaller stains can be removed using a cloth or moist sponge together with a little liquid detergent for delicate fabrics if necessary.
- **Only use the heating pad again once the heating pad and cover are completely dry.**

## 5. Storage

If you are not going to use the heating pad for some time, we recommend storing it in its original packaging in a dry place without weighing it down. Allow the heating pad to cool down beforehand.

## 6. Disposal

Please dispose of the blanket in accordance with the directive 2002/96/EG – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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